


# LactationHub™ nipple measuring tool

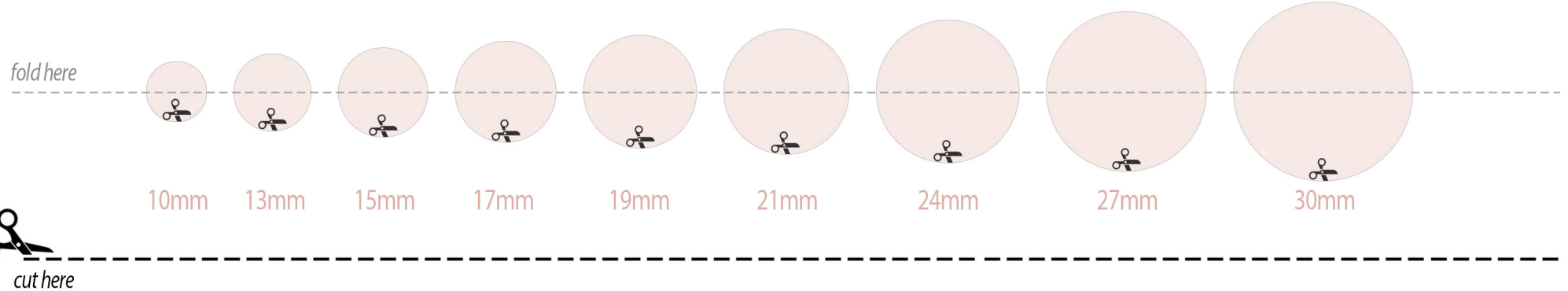

1. Fold the ruler along the fold line to easily cut the circles
1. Carefully cut, then unfold the ruler.
3. Open the page back up and then place each hole around your nipple.
4. Select the hole that fits your nipple most snugly, without any areola present

**IMPORTANT!** To ensure accurate measurements, make sure to print on:  
**LETTER SIZE paper** and **print to scale: Full Size (100%)**



scan to buy the best  
silicone flange inserts!



### Clear Comfort Silicone Inserts:

Add 1-2mm to your result



### Plastic flanges:

Add 0-1mm to your result



To check printing accuracy, place a US  
quarter right here to match the size.